

Zedlink - linking images to your desktop and mobile

Zedlink is a web based radiology viewing platform where you can view your patient's images and reports by connecting to the internet on your computer and navigating to: <https://zedlink.thexraygroup.com.au>

Prior to accessing Zedlink, you will need to be setup with a username and password. Your representative from The X-Ray Group will be able to assist you with this.

Enter username and password, click Logon



The screenshot shows the ZedLink login interface. At the top left is the 'the x-ray group' logo. In the top right corner, it displays 'Version: 1.3.8.1' and 'English (United States)'. The main heading is 'ZedLink' with the tagline 'linking images to your local desktop...'. Below this, there are two input fields: 'User ID' and 'Password'. A 'Login' button is positioned to the right of the password field. In the bottom left corner, there is a logo for 'powered by ZedTechnologies'.

When you access the Zedlink site, you will be prompted to enter your User ID and Password. When you have successfully logged into Zedlink, you can search for your patients reports and images.


To search for your patient enter surname, first name, the stored studies will be shown, to open the study double click on it.



The screenshot shows the 'Studies' page in the ZedLink application. At the top left is the 'the x-ray group' logo. A navigation bar contains 'Studies', 'Queues', and 'Admin'. The main heading is 'Studies'. Below this, there is a 'Partitions' section with a dropdown menu showing 'ZEDSVR1'. A 'Patient Name' search field is present with a magnifying glass icon. Below the search field are three buttons: 'View Images', 'View Details', and 'Delete'. At the bottom, it indicates '0 Studies Found'.



Note: Zedlink is not a PACS System It is a Footprintless viewer for displaying recent images.

Dicom tools are available in the top toolbar

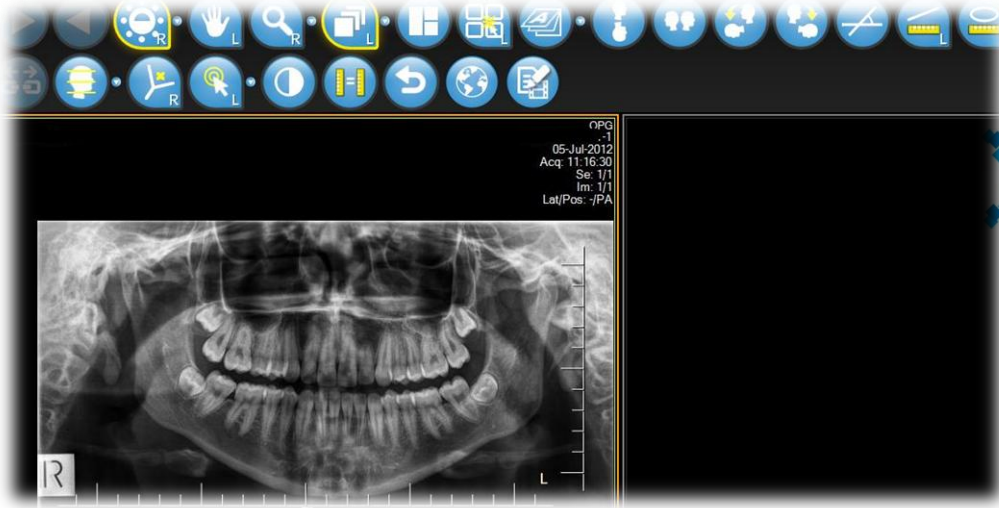

Zedlink is also available for iPhone and iPad, **FREE** from the Apple Apps Store.

Simply **Download XRG Mobile** By Zed Technologies

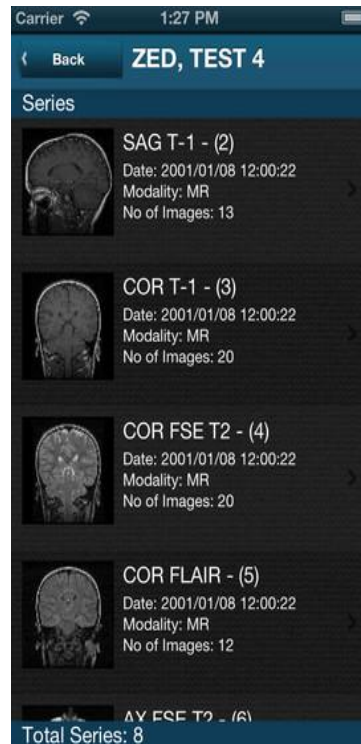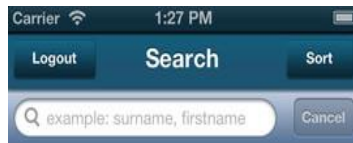

Note: XRG Mobile by Zed Technologies is not for diagnostic use